



**PRODUCT CODE: 255805**

**Histofix ® Substitute of Formaldehyde for clinical diagnosis**

(® Registered trade-mark of Panreac Química S.L.U.)for histology, fixing

---

TARIC 3822 00 00 00

**PHYSICAL DATA:** liquid, Clear, Colourless to yellow , Soluble in water. D 20/4 1,111 • B.P.: 85 - 104 °C • pH3,5 - 4,2 • Flash P.:66 °C •

**INSTRUCTIONS FOR USE:** Dilute before use. For regular fixations: Mix 1 L of Histofix® Substitute of Formaldehyde, 3.1 L of water and 0.9 L of absolute ethanol. For fixations with alcoholic effect: Mix 1 L of Histofix® Substitute of Formaldehyde, 2 L of water and 1.9 L of absolute ethanol.

**BIBLIOGRAPHY:**

**HAZARDOUS:**



H: H332 • H319 • H315 • H317 • H341 •

P: P201 • P202 • P261 • P264 • P271 • P272 • P280 • P281 • P302+P352 • P304+P340 • P305+P351+P338 • P308+P313 • P312 • P321 • P332+P313 • P333+P313 • P337+P313 • P362 • P363 • P405 • P501 •

**WEIGHT/VOLUME INFORMATION:** 1l~1,111 kg      1kg~0,9 l

**COMPOSITION:**

Glyoxal	15-25 %
Ethanol absolute	5-8 %
Acetic Acid glacial	<5 %
Methanol	<0,5 %

**SPECIFICATIONS:**

Identity :  
Identity  
pH

passes test  
3,5 - 4,25

Ed.: 5 . Vig.: 12.07.2017 .

Prod.: 255805