



PRODUCT CODE: 256284

Histofix ® marrow decalcifier for clinical diagnosis

(® Registered trade-mark of Panreac Química S.L.U.)

TARIC 3822 00 00 00

PHYSICAL DATA:

INSTRUCTIONS FOR USE: Submerge the specimen in Solution A (fixative) and let stand for a minimum of 2.5 hours. Rinse the fixed specimen with Ethanol 50% or 70% for 15 minutes. Submerge the specimen in Solution B (decalcifier) and let stand for a minimum of 3 hours. Check that the texture of the piece is suitable for further processing.

BIBLIOGRAPHY:

HAZARDOUS:



H: H331 • H311 • H301 • H373 • H412 •

P: P260 • P261 • P264 • P270 • P271 • P501 • P273 • P280 • P301+P310 • P302+P352 • P304+P340 • P311 • P312 • P314 • P321 • P322 • P330 • P361 • P363 • P403+P233 • P405 •

TRANSPORT REGULATIONS: UN: 2922 • ADR: 8(6.1)/II • IMDG: 8(6.1)/II • IATA: 8(6.1)/II • PAX: 851 • CAO: 855 • (E) •

SPECIFICATIONS:

Comprised of:
3x100 ml Solution A fixative
3x100 ml Solution B decalcifier

Ed.: 4 . Vig.: 11.12.2015 .

Prod.: 256284