



**PRODUCT CODE: 256876**

**Paraffin Cleaner for clinical diagnosis**

Microtomes cleaner used in the processing of human tissue, Sanitary product for in vitro diagnostic class A.

---

TARIC 3822 00 00 00

**PHYSICAL DATA:** liquid, Clear, Colourless, Soluble in water 0,1 g/l at 20 °C D 20/4 0,766 • M.P.: -10 °C • B.P.: 96 °C • Flash P.:57 °C • Vap. press. 19 hPa •

**INSTRUCTIONS FOR USE:** First clean the microtome with a fine brush, removing paraffin not adhered to the instrument. Spray the apparatus part with paraffin spray cleaner. Wipe gently with a cloth or fine paper until the dissolved paraffin residue has been removed. Allow to dry.

**BIBLIOGRAPHY:**

**HAZARDOUS:** VLA-EC (1-propanol) 400 ppm1.000 mg/m<sup>3</sup> VLA-ED (1-propanol) 200 ppm500 mg/m<sup>3</sup>



H: H226 • H318 • H304 • H413 •

P: P210 • P261 • P280 • P301+P310 • P303+P361+P353 • P305+P351+P338 • P310 • P331 • P403+P233 • P405 • P501 •

**TRANSPORT REGULATIONS:** UN: 1993 • ADR: 3/III • IMDG: 3/III • IATA: 3/III • PAX: 355 • CAO: 366 • (D/E) •

**WEIGHT/VOLUME INFORMATION:** 1l~0,766 kg 1kg~1,305 l

**COMPOSITION:**

Isoparaffin H

425 ml

1-Propanol

75 ml

**SPECIFICATIONS:**

Identity :

Identity

passes test

**Maximum limit of impurities**

Water (H<sub>2</sub>O)

0,3 %

Ed.: 6 . Vig.: 14.02.2018 .

Prod.: 256876