



**PRODUCT CODE: 254807**

**Kit for Fast Staining in Haematology (Fast Panoptic) for clinical diagnosis**

---

TARIC 3822 00 00 00

**PHYSICAL DATA:** Flash P.:12 °C •

**INSTRUCTIONS FOR USE:** Prepare the blood smear on the slide and allow to air dry. Submerge the slide in a receptacle with the Fixative for fast staining (Panoptic No. 1) 5 times for 1 second each time. Drain the excess liquid over filter paper. Submerge in another receptacle with the Eosin for fast staining (Panoptic No. 2) 5 times for 1 second each time. Drain. Submerge in another receptacle with the Blue for fast staining (Panoptic No. 3) 5 times for 1 second each time. Drain. Rinse the smear with Buffer solution, pH 7.2. Dry and examine under the microscope. Depending on the type and thickness of the sample, the immersion time in the dyes can be varied.

**BIBLIOGRAPHY:**

**HAZARDOUS:**



H: H225 • H331 • H311 • H301 • H370 •

P: P210 • P233 • P240 • P241 • P242 • P501 • P243 • P260 • P261 • P264 • P270 • P271 • P280 • P301+P310 • P302+P352 • P303+P361+P353 • P304+P340 • P307+P311 • P311 • P312 • P321 • P322 • P330 • P361 • P363 • P370+P378 • P403+P233 • P403+P235 • P405 •

**TRANSPORT REGULATIONS:** UN: 1992 • ADR: 3(6.1)/II • IMDG: 3(6.1)/II • IATA: 3(6.1)/II • PAX: 352 • CAO: 364 • (D/E) •

**SPECIFICATIONS:**

Comprised of:  
253998 Blue for fast staining (Panoptic No. 3)  
(1x500 ml)  
253999 Eosin for fast staining (Panoptic No. 2)  
(1x500 ml)  
254101 Fixing for fast staining (Panoptic No. 1)  
(1x500 ml)

Ed.: 4 . Vig.: 11.12.2015 .

Prod.: 254807